


Spenco® Earthbound Replacement Insoles

Cat. No. 38-625

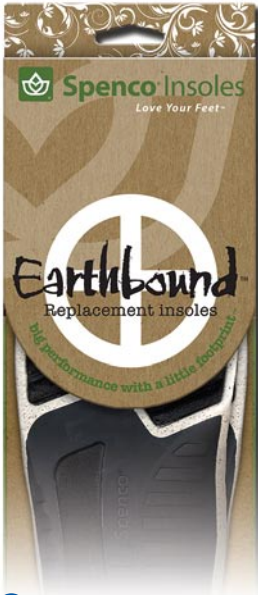

CONTAINS
55%
RECYCLED &
RENEWABLE
MATERIALS



Target Consumer:

Consumers seeking firmer support, motion control and a layer of cushioning in a more environmentally sustainable footbed.

Recommended For:

Increased shock absorption.
Advanced support for the arch and heel.
Improved motion control to reduce pronation and supination.

Best For Use In:

Hiking boots and shoes, athletic, tennis, basketball and training shoes with removable sockliners.

Components*:

Recycled EVA, cork, Polyurethane, Recycled Nylon 66, recycled polyester topcloth with *Ultra-Fresh Silpure* Anti-Microbial, Thickness at Heel: 8.50mm (3/8in), Thickness at Forefoot: 3.50mm (1/8in)

Instructions:

Remove the original insole from the shoe (some brands of shoes glue the insole to the shoe, but careful removal will not damage the shoe). Use the original insole as a guide to trace and trim the Spenco® Earthbound Replacement Insole. Insert the Spenco® Insoles into the shoes.

Care:

Hand wash in warm water. Air dry.

Sizing Chart:

Size	0	1	2	3	4	5	6
U.S. Women's	3/4	5/6	7/8	9/10	11/12		
U.S. Men's		6/7	8/9	10/11	12/13	14/15	
U.K.	2/3	4/5	5/6	7/8	9/10	11/12	13/14
European	32/34	36/38	38/40	40/42	42/44	44/46	46/48

Features & Benefits:

- 3-Pod™ Cushioning System** (patent pending) absorbs shock and helps prevent over pronation.
- PolySorb® Stability Cradle** made with recycled nylon 66 provides advanced arch support and deep heel cupping.
- Recycled Fabric with Ultra-Fresh Silpure Antimicrobial** helps prevent blisters and control odor.
- Metatarsal Arch Support** offloads pressure from the ball of foot to help relieve forefoot pain.
- Forefoot Crash Pad** provides extra cushioning under the ball of foot for a smooth transition to toe-off.
- Recycled EVA Cork** 65% EVA and 35% cork provides cushioning.

One-Year Unconditional Guarantee

Call toll-free: 1-800-877-3626
In Canada: 1-800-387-9538

www.spenco.com



*Measurements are approximate and vary by size.